

MedReady™



medreadyinc.net



- *Timed-alarm system provides next dosage automatically*
- *Operates by easy-to-use microchip technology*
- *Holds up to 7 days of 4 times a day dosing*
- *Costs less than similar devices*
- *Flashing light available*

Automated Medication Dispenser




MedReady, Inc.
2103 Border Ave.
Torrance, CA 90501
310-328-7557
Sales@MedReady.net

*One Less Argument • More Independence
Feel Better • Great Price*

FEATURES

Basic 1600

Large 1/2" L.E.D. displays the time and aids setting alarms.

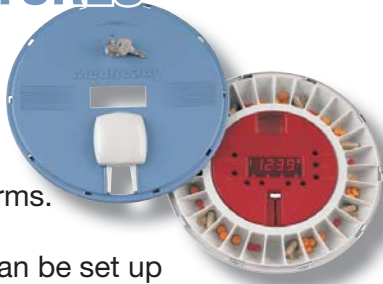

- Audible alarm can be set up to 4 times a day:
 - ◆ "Time for medicine" alarm beeps 5 seconds on, 1 second off continuously until the medication door is slid open and the "off" position is activated.
 - ◆ Note: Disabling alarm requires the medication door to be slid open.
 - ◆ Programmable alarm sounds for 30, 45, 60, or 250 minutes, as user selects
 - ◆ Powerful alarm sounds similar in tone and volume to a beeping microwave alarm but alarms on for 5 seconds and off for one second until disabled.
 - ◆ Audible alarm is 90dB, 2800-3000Hz. (**1600LF** is approximately 400Hz)
- Only the most recent medication dose is available. Pill tray rotates to the next dose when the next audible alarm sounds. If a patient misses a dose, the pill dispenser denies access to the missed dose.

MedReady™


Automated Medication Dispenser

- MedReady holds 28 doses of dry medication such as pills or capsules.
 - ◆ 4 Doses a day for 7 days
 - ◆ 3 Doses a day for 9 days
 - ◆ 2 Doses a day for 14 days
 - ◆ 1 Dose a day for 28 days
- Alarm in pill dispenser is as easy to set as a digital clock.
- Medication is under lock and key, not just a latch.
- Pill compartments are designed to fit adult fingers and "ramped" for easy removal of medication.
- Compartment divisions keep medications in place even if the unit is turned upside-down.
- MedReady should remain plugged into electric power, although there is a 48 hour rechargeable battery back-up for travel or power failures. MedReady should not be used in wet or very damp environments.
- "OFF" feature for long term disabling of the dispenser.
- "Early dosage" feature allows taking dosage with client.

OPTIONS

for

MedReady Medication Dispenser 1600

1. **5310 Extra Pill Tray** with clear plastic lid. Preload an additional tray for convenience.
2. **1600FL Alarm With Flashing Light** for hearing impaired. Audible alarm PLUS Flashing Light. 
3. **1600LF Lower Frequency Alarm** is louder for those losing high frequency hearing.
4. **1650 Remote Monitoring:** Simply plug the added telephone cable connector from the MedReady unit to the phone line.
 - ◆ Patient only needs phone line. Caregiver accesses secure web site to review actual medication times.
 - ◆ Monthly service plan.
 - ◆ Calls up to 3 phone numbers, leaving messages, and sends e-mail, if dose is not taken.
 - ◆ A record of the medication compliance and/or non-compliance is available on the **medready.net** web site.

