

## Lock Assembly Instructions

- 1) The lock fits into the lid with the hole in the threads facing down (towards the door)
- 2) The Key is in the lock and is in the horizontal position
- 3) The 3/4 washer on the underside is next and it points towards the 4PM position
- 4) Snug large nut (do not over tighten)
- 5) Place tab on the bolt with beveled edge on the right side facing you and pointing down.
- 6) Place lock washer over bolt
- 7) Tighten nut while holding the key on the other side

### Note –

Assembly instructions are stated while looking at the backside of the lid with the lock at the top and the door at the bottom.

When in the open position, the tab is pointing straight down towards the door or 6:00 position.

When in the locked position the tab is pointing to the left or the 9:00 position

**Lock assembly video can be found at:  
[medreadyinc.net](http://medreadyinc.net)**

MedReady Inc.

