

**IMPORTANT: Replace all the parts that are in this shipment, and discard the old parts. These new parts are slightly different and must be used together for the MedReady to function.**

**To replace the gear set you will need a small Phillips head screwdriver and a Q-tip with one end cut off.**

- 1- Remove the blue lid and the tray.
- 2- Remove the 4 screws from the red cover using a Phillips head screwdriver. Note that there is not much turning force required.
- 3- Making sure that the white bar is up, remove the red cover. It will resist a little in the right back.
- 4- The gears will be on the left side. Look at the way the gear is sitting on the risers. Remove the gears.
- 5- Place the new gear set in the set way as the old.
- 6- Take a Q-tip with one end cut off and insert through the lower left screw hole from the bottom.
- 7- Place the red cover in position using the Q-tip as a guide. Care must be taken to align the vertical gear. It must slide in a hole in the red cover. Once the red cover is in place, gently attempt to rotate the gear protruding through the red cover. If it rotates freely the vertical gear is not in the hole.
- 8- Replace the four screws. Tighten them gently. This is plastic and can be stripped.

View replacement video at: <http://www.medreadyinc.net/instructions.html>

Best regards,

MedReady Inc.

