



MedReady™

Available with GSM cellular
compliance monitoring



medreadyinc.net

Automated Medication Dispenser

- *Timed-alarm system provides next dosage automatically*
- *Holds up to 7 days of 4 times a day dosing*
- *MedReady's functions and capabilities provide the best value in the medication dispensing marketplace*
- *The cellular dispenser has the capability to communicate compliance events via cell towers. No phone line or internet connection is necessary at the dispenser location.*
- *Check compliance history, change dispensing times, and update all device settings via the www.medready.net website.*

MedReady, Inc.
2103 Border Ave.
Torrance, CA 90501
310-328-7557
Sales@MedReady.net

*One Less Argument • More Independence
Feel Better • Great Price*



FEATURES

Large 1/2" L.E.D. displays the time and aids setting alarms.



- Medication regimen can dose up to 4 times per day:
 - ♦ "Time for medicine" alarm beeps 5 seconds on, 1 second off continuously until the medication door is slid open and the "off" position is activated.
 - ♦ Note: Disabling alarm requires the medication door to be slid open.
 - ♦ Programmable alarm sounds for 3, 10, 20, 30, ...250 minutes, as user selects, and can chirp each minute thereafter until the dose is taken or the next one is available.
 - ♦ Powerful, 90dB, 400Hz. audible alarm is designed for maximum effectiveness.
 - ♦ Audible and visual alarms can be independently turned on or off.
- Only the most recent medication dose is available. Pill tray rotates to the next dose when the next audible alarm sounds. If a patient misses a dose, the pill dispenser denies access to the missed dose.

MedReady™

Automated Medication Dispenser
Available with GSM cellular compliance monitoring

- MedReady holds 28 doses of dry medication such as pills or capsules.
 - ♦ 4 Doses a day for 7 days
 - ♦ 3 Doses a day for 9 days
 - ♦ 2 Doses a day for 14 days
 - ♦ 1 Dose a day for 28 days
- Alarm in pill dispenser is as easy to set as a digital clock.
- Medication is under lock and key, and is tamper resistant.
- Pill compartments are "ramped" for easy removal and can fit up to 9 regular m&m's.
- Compartment divisions keep medications in place even if the unit is turned upside-down.
- MedReady should remain plugged into electric power, although there is a 48 hour rechargeable battery back-up for travel or power failures. MedReady should not be used in wet or very damp environments.
- "OFF" feature for long term disabling of the dispenser.
- "Early dosage" feature allows removal of medication prior to scheduled times.

- Cellular GSM Communication from the MedReady dispensers to our servers.
 - ♦ Compliance notifications via email, text, and automated phone calls.
 - ♦ All medication compliance events are communicated in real time.
 - ♦ Monthly service plan includes monitoring and MedReady.net compliance website access.
- Manage MedReady online
 - ♦ Set dispensing times, change device settings, trigger early doses view detailed reports.

Remaining Doses: 26

Days Remaining: 6

ALARM TIME: Friday 03/15/13

ALARM TIME	DOSE TIME
06:00	06:10
09:00	09:05
15:00	16:10
18:00	18:15

Alarm	Time	Active?	Meds	Action
Alarm 1	12:46pm	Active: Yes		Edit
Alarm 2	12:47pm	Active: Yes		Edit
Alarm 3	12:48pm	Active: Yes		Edit
Alarm 4	12:50pm	Active: Yes		Edit

Other Functions

Alarm Test Early Dose Cancel Early Dose

Send Send Send

OPTIONS

1. **MedReady Monitored MR-357** using GSM Cellular Enabled.
2. **MedReady Monitored 1750** using Plain Old Telephone Service (POTS)
3. **FL - Alarm With Flashing Light** for hearing impaired. Audible alarm PLUS Flashing Light.
4. **5310 Extra Pill Tray** with clear plastic lid. Preload an additional tray for convenience.

